


Update Description

The service pack CEREC 3D SW V3.83 updates existing V3.80 installations and contains the following changes:

■ **Improved Pre-Positioning Tool**

In case of unsatisfactory initial proposals, if the user chooses "Pre-Positioning" from the menu "Design", an improved version of the non-positioned restoration is displayed, an expansion on the "wireframe" mode from V3.65 ("Undo" function). In this mode, the user can manually choose the desired position and rotation for the restoration. The Pre-Positioning Tool is available from the Design step for all full-anatomical restorations and design modes.

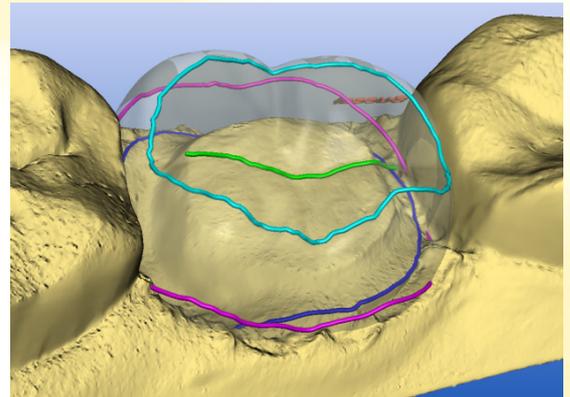

■ **Further development of Biogeneric algorithms**

Initial proposals are more natural, especially in the anterior.

■ **Smooth Tool with more definition**

Working with the Smooth tool has a more profound effect on restoration surfaces, similar to the effect in V3.65.

■ **Additional materials for thin veneers**

CEREC Blocs, CEREC Blocs PC and VITA Mark II have been approved for thin veneers between 300 and 450 μm thickness.

■ **Bridge support for exports to GALILEOS Implant**

Bridge restorations are now supported for export to GALILEOS Implant.

■ **Various minor improvements**

An existing CEREC 3D SW V3.80 installation is required for this service pack. We strongly recommend this update to all V3.80 users.